



11 ACB • 12 ACB

Atraumatic Clockwise Blue  
Preloaded Capsular Tension Ring

**JET** RING



**MEDI**  **NTUR**  
Vision of Expertise

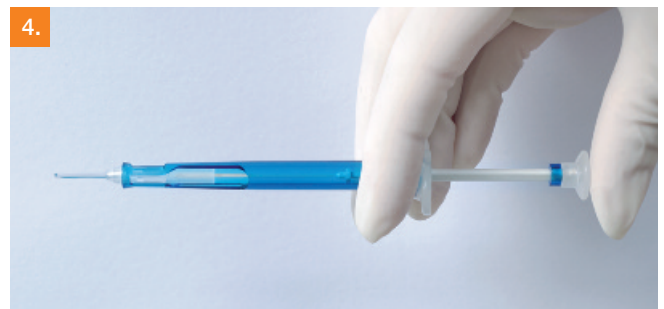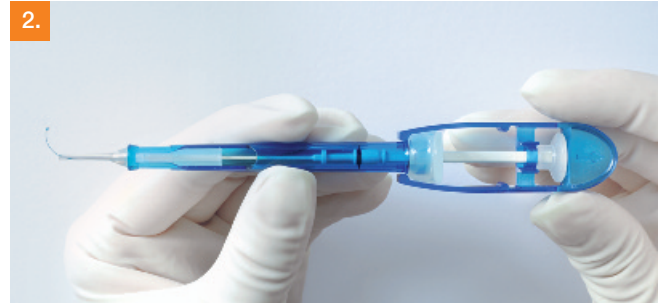

1. Gently remove the injector from the blister.
2. Carefully pull the safety clamp together with the pusher until the stop.
3. Remove the safety clamp from the pusher in a lateral direction.
4. Completely retract the preloaded CTR into the injector by further pulling the pusher until the end.
5. Insert the injector tip into the capsular bag at 6 o'clock and release the ring along the capsular equator clockwise by gently pushing the rod. Make sure the ring is sliding smoothly around. At the end the ring will be released from the hook. Retract the empty hook into the injector and remove it from the eye.

### Technical data

<b>Injector</b>	Preloaded - single use
<b>Ring Overall Diameter</b>	11 mm (uncompressed 13 mm) 12 mm (uncompressed 14 mm)
<b>Material of the ring</b>	PMMA blue
<b>Type of ring</b>	Atraumatic
<b>Sterilization</b>	Gamma irradiation

LA-IMCJREN 02 201207